

Guidance to be read in conjunction with user manual.

1: New Handles:

Saviour Stretchers now feature new ergonomic handles to make extended duration carries more comfortable. (fig 1)





2: Soft Wrist Cuffs;

To assist with upper limb management in unconscious casualties, the blue and yellow chest cross straps now have built-in wrist cuffs. (fig 2)

These can be easily slid up and down the strap via the black buckle to achieve the required position. (fig 3)

The soft velcro binds around the wrist to hold the upper limbs in place. (fig 4 & 5)

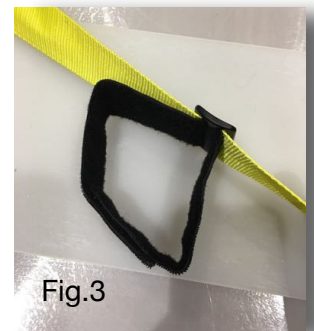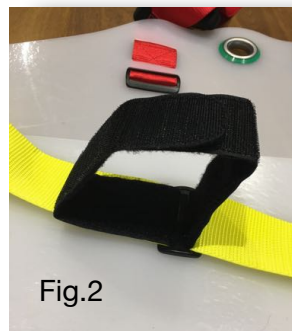

The wrist can be secured to the strap nearest to them, or the upper limbs can be "crossed" and wrist secured to the opposite side, dependent on which offers most comfort. (fig 6)

