

Saviour Tactical Stretcher + Lucas 2 Chest Compression System

Saviour Medical are proud to work with Physio Control to bring a perfect solution for moving patients when fitted with a LUCAS. The Saviour Tactical stretcher stabilises the LUCAS on the patient whilst encapsulating the patient making it easy to transport them. Saviour allows for easy evacuation from any location whilst the LUCAS maintains constant CPR. The Saviour Tactical stretcher can be lifted, dragged, lowered and vertically hauled. Combining LUCAS and Saviour provides the safest and most effective evacuation option for both patient and rescuer.

- **Head Hugger system and double central skin protect and immobilise spine.**
- **Acts as an effective pelvic splint**
- **Xray translucent and CT friendly**
- **All parts independently replaceable for cost effectiveness**
- **Easy to decontaminate making it multi not single use.**




USE OF THE DRAG HARNESS

There are two drag harnesses, one at the head end and one at the foot end. On removing the protective sheath, a tape loop is released that is permanently attached through metal eyelets on the stretcher.

Placing the loop over the head and around the shoulders will allow the rescuer to drag or pull the casualty in the required direction while having their hands free to carry out other duties.

The same dragging or pulling technique is used when moving the casualty head or feet first.

The drag harnesses can be utilised to lower a patient down stairs safely without endangering the rescuers back.

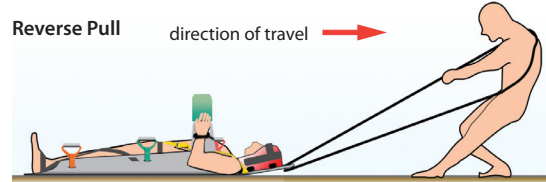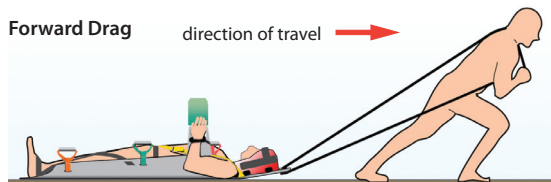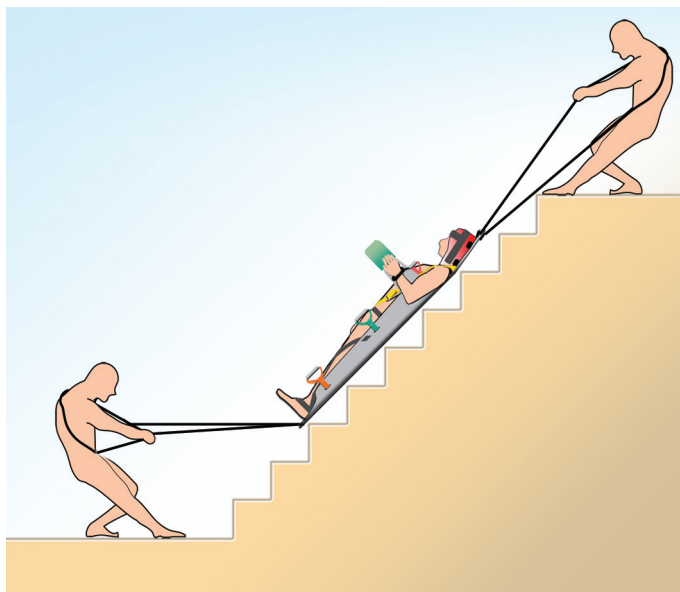

VERTICAL HAUL



The Saviour Tactical stretcher is rated for vertical lifting from the top eyelet (not the drag harness). As this only effects a single point of contact with the casualty, a second safety line secured to the casualty is always recommended.

In order to make the vertical haul easier, it is also recommended that both the upper and lower drag harnesses be removed from the stretcher before the lift is made.


The Saviour Tactical stretcher only weighs 5kg and is easy to stow.




CARRYING THE CASUALTY

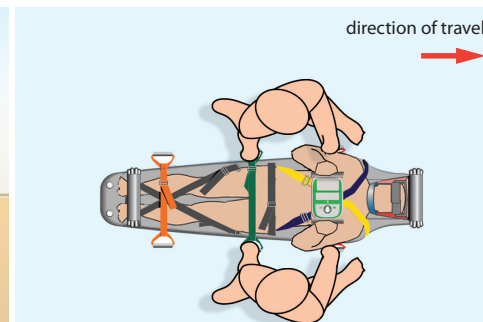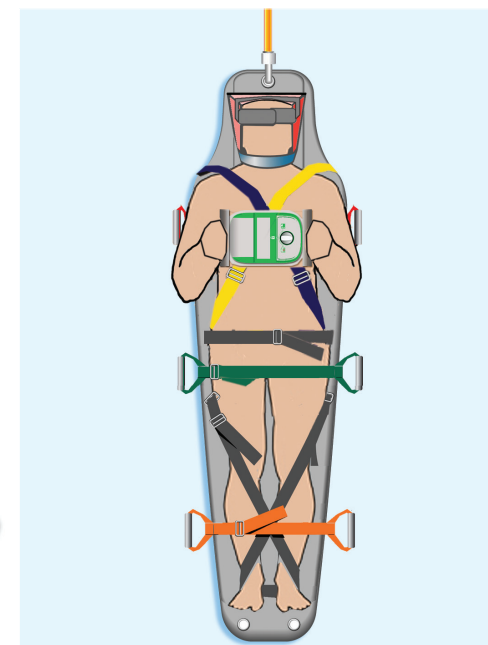


2 man carry

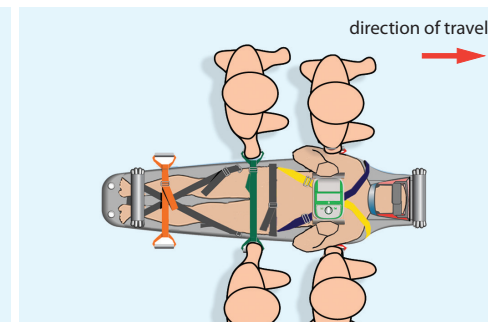

4 man carry

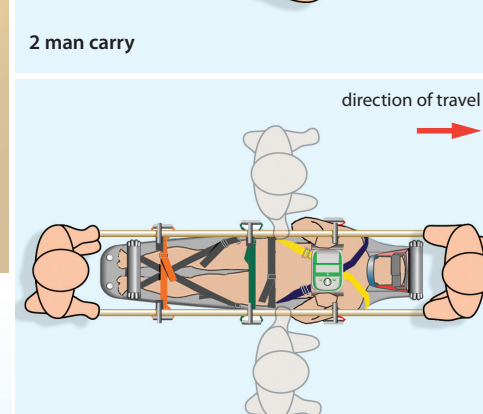

2 (or 4) man pole carry for long distance carrying

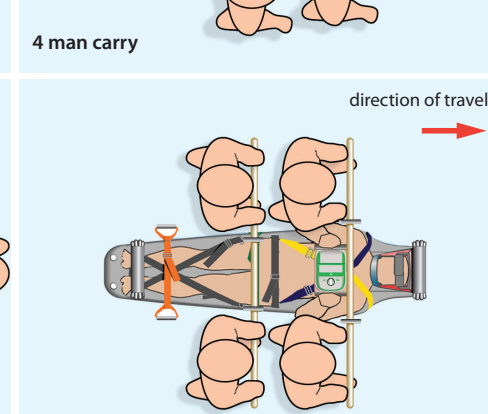

4 man side to side pole carry for heavy casualties